

FABRIC HUSH CURTAIN® LAUNDRY INSTRUCTIONS

PREPARE THE HUSH CURTAIN® FOR LAUNDERING

- Unzip each HUSH Curtain® panel section from one another.
- Remove the curtains from the track leaving the roller carries in the track.
- Safely transport to the laundry facility.



DO NOT LAUNDER THE ACOUSTIC BAFFLES

- Prior to laundering, carefully remove each inner HUSH Curtain® Acoustic baffle from the pockets of the HUSH Curtain® shell.
- Store the acoustic baffles in a clean, sanitary area or put into a clean curtain shell and hang during the curtain shell cleaning procedure.
- Use a disinfectant wipe (low concentration bleach based, e.g. Clorox Dispatch™) if needed.
- Allow to dry completely prior to re-insertion into the HUSH Fabric Panel.

CLEANING

Launder and dry the curtain shells according to the HUSH Curtain® laundry instructions below unless otherwise provided based upon the shell panel fabric selected.

- Machine wash with mild soap or detergent, on a synthetic setting with high water levels and mild agitation.
- Water temperature should not exceed 160°F.
- Do not use bleach or fabric softener.
- Wash with like materials. Extract gently.
- Air, cabinet, or tumble-dry on a synthetic cycle at a low temperature.
- Do not exceed 140°F for more than 3-5 minutes.
- Hang or fold immediately after drying.
- **Failure to follow exact laundry directions and temperatures will void the product warranty.**

PREPARATION FOR RE-INSTALLATION

- Simply re-insert each acoustic baffle into the patented curtain shell pockets from the top, open end.
- Once re-installed in the cubicle, zip together each HUSH Curtain® section starting at the top.

REMOVEABLE ACOUSTICAL INSERTS

Fabric Hush Curtain panels only

IMPORTANT: FAILURE TO FOLLOW THESE INSTRUCTIONS WILL VOID WARRANTY.



1
Prior to laundering, carefully remove each inner HUSH Curtain® Acoustic baffle from the pockets of the HUSH Curtain® shell.



2
Store the acoustic baffles in a clean, sanitary area during the curtain shell cleaning procedure.

Do not launder the acoustic baffles.



3
Use a disinfectant wipe (low concentration bleach based, e.g. Clorox Dispatch™) if needed.



4
Allow to dry completely prior to re-insertion into a clean HUSH Curtain® fabric shell.

DISINFECTING CLEAN-IN-PLACE™ CURTAINS

LAUNDERING THE ZIP TOP MESH



- Machine wash with mild soap or detergent, on a synthetic setting with high water levels and mild agitation.
- Water temperature should not exceed 160F°.
- Extract gently from washer to not tear mesh.
- Air, cabinet, or tumble dry on a synthetic cycle at a low temperature. Not to exceed 140F° for more than 3-5 minutes.

PREPARING THE CLEAN-IN-PLACE™ HUSH CURTAIN® FOR CLEANING

- Unzip each vinyl Clean-In-Place™ panel from one another using the side zipper (if applicable).
- Unzip the zip top mesh top from vinyl Clean-In-Place™ panel.
- Launder and dry the zip top mesh according to the HUSH Curtain® laundry instructions below.
- Use an approved disinfecting cleaning solution on vinyl Clean-In-Place™ panel.

DISINFECTING CLEAN-IN-PLACE™ VINYL PANELS

**Ask us for the most current list of approved cleaners from fabric manufacturer*

- Use only an approved disinfecting cleaners on all vinyl Clean-In-Place™ panels.
- Using a wiping motion, thoroughly disinfect both sides of the panel based on the cleaning solution instructions.
- Do not soak Clean-In-Place™ panels in disinfectant.
- Allow the vinyl material to properly dry based on the cleaning solution instructions.
- **Failure to follow directions will void the product warranty.**



PREPARATION FOR RE-INSTALLATION

- Zip the laundered zip top mesh top back on the Clean-In-Place™ vinyl panel.
- Using the side zippers, zip the Clean-In-Place™ panels back together.

INSTALLATION DIRECTIONS


HELPFUL MATERIALS & SUPPLIES TO HAVE ON HAND

- Additional roller carriers (we recommend new roller carriers are installed for ease of movement)
- Flathead screwdriver to add extra carriers to track
- Silicon grease or spray to lubricate track
- Ladder

SOME HELPFUL ADVICE TO INSTALL HUSH CURTAIN®

HUSH Curtain® easily installs by hanging on the already existing hospital curtain track.

- It is the responsibility of the facility to install the cubicle track per the manufacturer's direction regarding ceiling attachment and support.
- Simply remove the hospital curtain and hang HUSH Curtain® in its place.
- Once all panels are hanging, zip them together to create one seamless curtain.

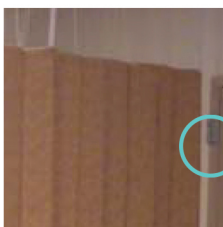


TIP #1

- Start hanging panels with the roller carriers closest to the wall.
- Make sure the panel edge **without the zipper pull** is adjacent to the wall.



TIP #2

- One end of each panel has a zipper pull, while the other end has only zipper teeth.
- Place the end with only teeth against the wall when starting to install. That will ensure the zipper pull is available to attach the next panel.
- When hanging each curtain, make sure that you coordinate the teeth and the zipper pull with the previous curtain, so when install is done, they are able to zip together.

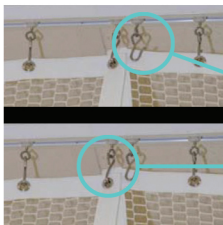

TIP #3

- Position one free-floating roller carrier between the overlapping grommets so one is available if the panels ever need to separate.
- Overlap the end grommet of each panel onto one roller carrier for ease of movement.

HUSH CURTAIN® WARRANTY

HUSH Curtain® Acoustic Privacy Panels warranties the materials for one year from receipt of the order against defects under normal handling and compliance with all fabric care instructions.

Failure to follow all care instructions voids the warranty.

